


# Deliver a Bolus


for the t:slim X2™ Insulin Pump

The instructions below are provided as a reference tool for caregivers who are already familiar with the use of an insulin pump and with insulin therapy in general. Not all screens are shown. For more detailed information on the operation of Tandem's insulin pumps, please visit [www.tandemdiabetes.com/support](http://www.tandemdiabetes.com/support).

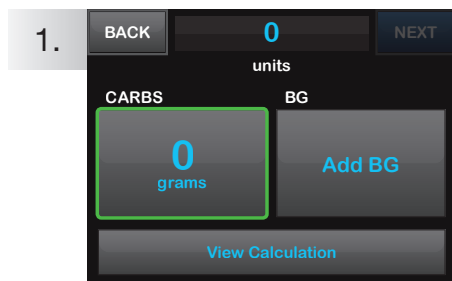


1. Tap **0 grams** to enter the carbs for your bolus.


**NOTE:** If this button reads "units," the carb feature is turned off in the active profile.

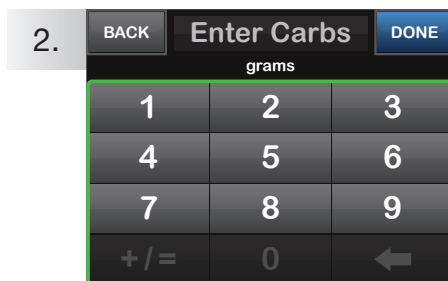


2. Enter desired value. Be sure "grams" is displayed above keypad for food boluses. Tap **DONE** to continue.

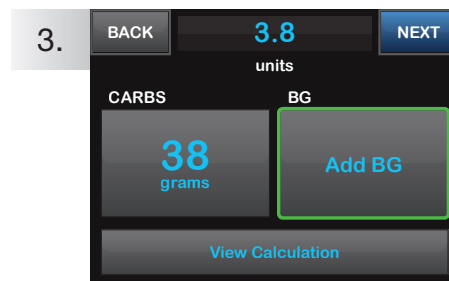


3. Tap **Add BG** to enter your blood glucose.

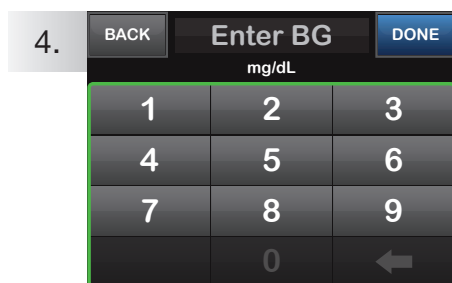


4. Enter desired value. Be sure "mg/dL" is displayed above keypad when entering BG values.

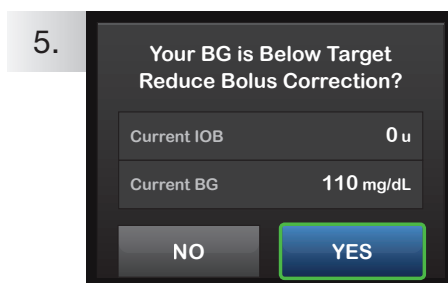

5. If a BG is entered that is below the target, but above 70 mg/dL, you will be offered the option to reduce the bolus amount. To accept that reduction tap **YES**; otherwise, tap **NO**.

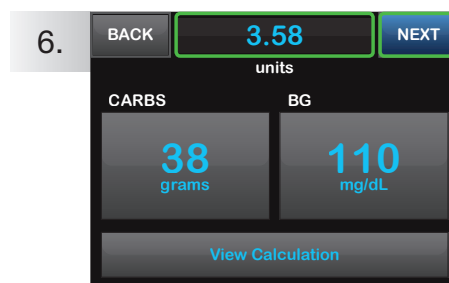


6. Tap **NEXT** to continue. Tap the calculated units value to manually adjust recommended dose.

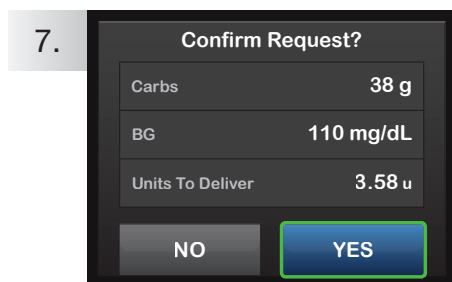


7. Verify the dose and tap **YES** to confirm.

**NOTE:** Calculations above are based on preset insulin-to-carb ratios and correction factors, which may be set in Personal Profiles.

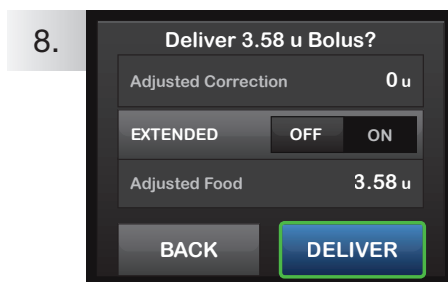

8. Tap **DELIVER** to deliver the food bolus immediately.

The **BOLUS INITIATED** screen will appear to confirm delivery has started.

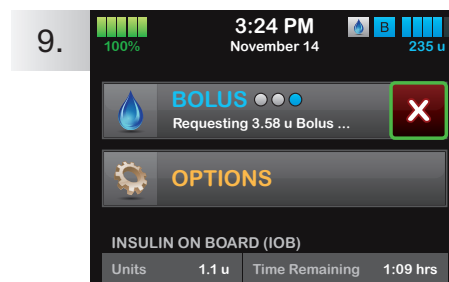

9. To cancel the undelivered portion of the bolus, tap the white X on the **BOLUS** button on the Home Screen, then tap **YES** to confirm canceled bolus.

